Recaro Monza CFX Reha

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Dear customer,

You have decided in favour of our product and we would like to thank you for the trust you have placed in us. To ensure that the handling of your new product is safe, practical and convenient for you and the user, please read these instructions for use first. This will help you to avoid damage and operating errors.

operating errors. Please ensure that these instructions for use remain with the product and are available to every user.

This second part of the instructions for use describes the technical rehab equipment, which may differ from the consumer product.

If you have any questions, please contact your specialist dealer or us directly.

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Phone: +49 (0) 4761 8860

www.thomashilfen.com

We reserve the right to make technical changes. This may lead to deviations between your product and the representation in media.

Table of contents

1. Foreword / Manufacturer	38
2. Operating instructions Consumer car seat	36
3. Rehab car seat illustrations	18
4. Labels / model identification	50
5. Remarks.	51
5.1 General remarks	51
5.2 Safety instructions	51
6. Intended Purpose.	53
6.1 Intended Use / Application Areas	53
6.2 Indication / target patient group, intended users	53
6.3 Risks of use and contraindications	53
7. Product description	54
7.1 Scope of delivery	54
7.2 Basic equipment.	54
7.3 Commissioning.	54
7.4 User training.	54

8. Adjustment options / performance features	55
9. Accessories	56
10. Technical data	60
11. Material specifications	60
12. Cleaning, maintenance and repair	61
12.1 Cleaning, care instructions and disinfection in everyday life	61
12.2 Maintenance and inspection by the distributor	62
12.3 Maintenance schedule – annual inspection	62
12.4 Spare Parts	63
12.5 Storage	63
12.6 Disposal	63
13. EU Declaration of Conformity	63
14. Warranty	63
15. Service life / Re-use	64
16. Distributors	65

HOMOLOGATION

Monza CFX

UN R129/03 i-Size Booster Seat

Height: 100-150 cm

Age: ca. 3,5-12 Years

Only suitable for vehicle seats with automatic three-point belts, approved in accordance UN regulation no. R16 or a comparable standard.

Child seat of the UN R129/03 Category „i-Size booster seat“ for stature height 100-150 cm.

If the i-Size booster seat is used for children taller than 135 cm it may not fit on vehicle seats declared i-Size compatible.

SAFETY IS A MATTER OF TRUST

Thank you for choosing RECARO. Thanks to our unique experience RECARO has been revolutionising car, aircraft and racing car seating for over 100 years. This unrivaled expertise feeds into the development of all of our child safety systems. Our aim is to offer you products which set new standards in terms of design, functionality, comfort and safety. We can assure you that safety, comfort and ease of use were our highest priorities when developing the child seat. This product is manufactured under exceptional quality control and meets the strictest safety requirements.

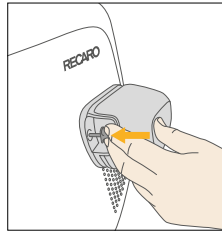
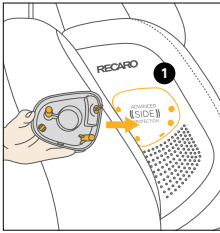
We wish you lots of fun and a safe journey every time.
Your RECARO Kids team

WARNING! In order to keep your child properly protected it is absolutely essential that you use the child seat as described in this manual.

NOTE! Please don't dispose of the instruction manual and keep it save for future references.

NOTE! Due to country-specific requirements (e.g. colour labelling on the child seat), the product features may vary in their external appearance. This, however, does not affect the correct functioning of the product.

NOTE! The three-point belt must only be passed through the designated routings. The belt routings are described in detail in this manual and are marked in green on the child seat.

FIRST INSTALLATION

Insert the “Advanced Side Protection” element (ASP) door-sided of the shoulder rest into the appropriate installation holes **1**. Press the pin inwards to lock the ASP.

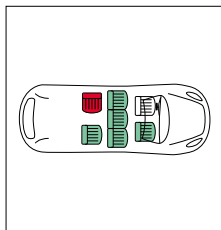
WARNING! Make sure to read, understand and follow the instruction manual.

NOTE! Please don't dispose of the instruction manual and keep it save for future references.

NOTICE! Even without the side protectors, the seat provides sufficient protection against side impact. If the vehicle provides sufficient space, we recommend installing the supplied ASP elements on the door side.

PROTECTING THE VEHICLE



Traces of use and/or discoloration can appear on some vehicle seats made from delicate materials (e.g. velour, leather, etc.) if car seats are used. You can avoid this by using, e.g. the RECARO Car Seat Protector. In this context also see our cleaning directions. It is essential that these are followed before the first use of the child seat.


CORRECT POSITION IN THE VEHICLE

This is an i-Size booster seat Enhanced Child Restraint System. It is approved according to UN Regulation No. 129, for use primarily in „i-Size seating positions“ as indicated by vehicle manufacturers in the vehicle user's manual. If in doubt, consult either the Enhanced Child Restraint System manufacturer or the retailer.

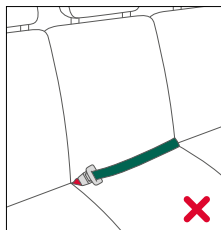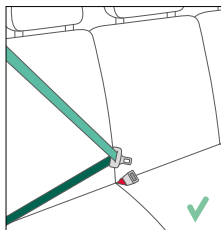IMPORTANT: For a complete list of cars, please visit <https://www.recaro-kids.com/pages/car-compatibility>

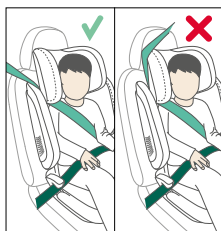
WARNING! This i-Size booster seat is not suitable for use with a two-point or lap belt. If the seat is secured using a two-point belt, an accident may result in severe or even fatal injuries to the child.

WARNING! The diagonal belt must run back at a recline and must never run forward to the upper belt point in your vehicle. If it is not possible to adjust the seat in this way, e.g. by pushing the child seat forward or using it on another seat, this child seat is not suitable for your vehicle. Contact the child seat manufacturer if in doubt. Luggage or other objects present in the vehicle that may cause injury in the event of an accident must always be firmly secured. Otherwise, they may be thrown, causing fatal injuries.

WARNING! The child seat may not be used on vehicle seats facing to the side of the direction of travel.

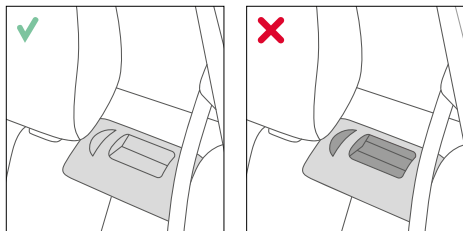WARNING! Never leave your child unattended in the car. The plastic parts in the child restraint system heat up in sun and the child may sustain burns. Protect your child and the child seat from direct sun exposure (e.g. by covering it with a light colored cloth).


SAFETY IN THE VEHICLE

WARNING! The child seat must always be secured correctly with the vehicle belt in the vehicle even when not in use. In case of an emergency brake or accident, an unsecured child seat may injure other passengers or yourself.

In order to guarantee the best possible safety for all passengers make sure that:

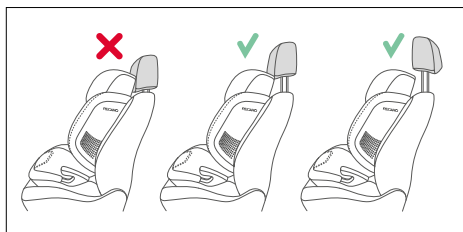- foldable backrests in the vehicle are locked in their upright position
- when installing the child seat on the front passenger seat, adjust the vehicle seat in rearmost position
- you properly secure all objects likely to cause injury in the case of an accident
- all passengers in the vehicle are buckled up





INSTALLING THE CHILD SEAT IN THE VEHICLE

Place the child seat on the appropriate seat in the vehicle:

- Always ensure that the entire surface of the backrest of the child seat is placed fully against the backrest of the vehicle seat
- If the headrest of the vehicle is in the way, pull it out all the way or remove it completely (for exceptions to this, please refer to the section "CORRECT POSITION IN THE VEHICLE")

WARNING! The full surface of the backrest of the child seat must be against the backrest of the vehicle.

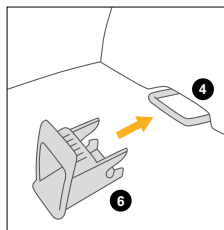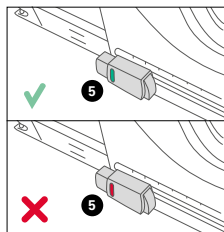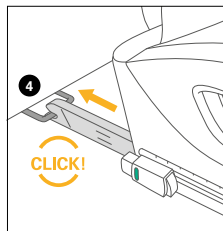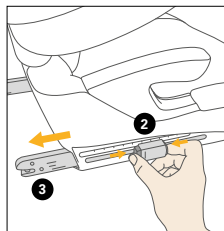WARNING! Do not use any load bearing contact points other than those described in the instructions and marked in the child restraint.

INSTALLING THE CHILD SEAT WITH ISOFIX ATTACHMENTS

INSTALLATION

Using the ISOFIX attachments, you can fix the child seat to the vehicle, thereby increasing your child's safety. ISOFIX attachments secure the seat when it is not in use. The child still needs to be secured with the three-point belt of your vehicle.

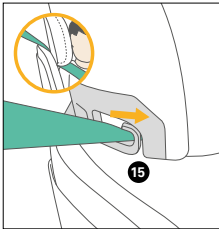
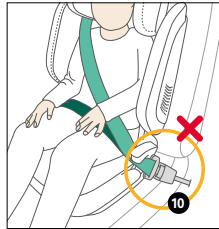
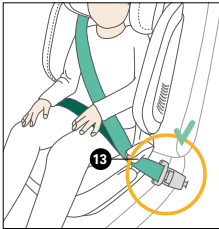
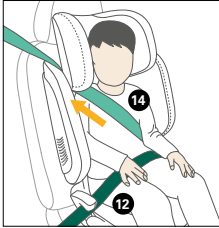
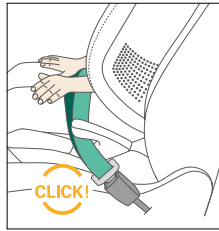
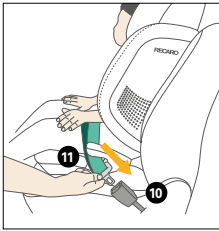
NOTE! The ISOFIX anchorage points **4** are two metal round horizontal bars for each seating position and are positioned between the backrest and the seating surface of the vehicle seat. In case of doubt, please refer to the driver's manual of your vehicle.

- Push both ISOFIX attachments **3** out as far as they will go. To do this press both buttons on the adjustment button **2**.
- Push the two ISOFIX attachments **3** into the ISOFIX anchorage points **4** until these lock into place with an audible 'CLICK'.
- Ensure that the child seat is secure by trying to pull it out
- The safety indicator **5** must now be green on both adjustment buttons.

Push the child seat backwards until the full surface of the backrest of the child seat is against the backrest of the vehicle.

NOTE! Are the ISOFIX anchorage points **4** hard to reach in your vehicle, you can use the supplied ISOFIX guides **6**.

- Insert the supplied ISOFIX guides **6** with the longer strap upwards into the two ISOFIX anchorage points **4**. In some vehicles, it is better to mount the ISOFIX guides **6** in the opposite direction

STRAPPING WITH VEHICLE SEAT BELT

Place your child in the child seat. Pull the three-point belt right out and feed it along the belt buckle **10** in front of your child.

WARNING! Never twist the belt.

Insert the belt tongue **11** into the belt buckle **10**. It must lock into place with an audible “CLICK!”. Next, place the lap belt **12** in the lower green belt guide **13** of the child seat. Now pull the diagonal belt **14** firmly to tighten the lap belt **12**. The tighter the belt sits, the better it provides protection from injury.

NOTE! On the side of the belt buckle **10**, the diagonal belt **14** and the lap belt **12** must be inserted together in the lower belt guide **13**.

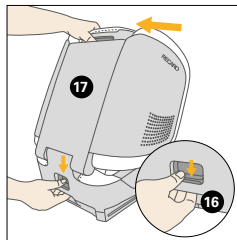
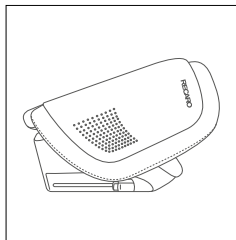
WARNING! The vehicle belt buckle **10** must never cross the arm rest. If the belt system is too long, the child seat is not suitable for use in this position in the vehicle. In case of doubt, please consult the manufacturer of your vehicle.

The lap belt **12** should have been fed through the lower belt guides on both sides of the seat cushion.

NOTE! Teach your child from the start to always ensure that the belt is tight and, if necessary, how to tighten the belt themselves.

WARNING! The lap belt **12** must run as low as possible on both sides along your child’s groin to have optimum effect in the event of an accident.




Now feed the diagonal belt **14** through the green upper belt guide **15** in the headrest until it is inside the belt guide. Ensure that the diagonal belt runs between the outside of the shoulder and the neck of your child. If necessary, adjust the height of the headrest **7** to change the position of the belt. The height of the headrest can still be adjusted while in the car.



UNFOLDING THE CHILD SEAT

Press down the button at the bottom of the backrest **17** and bring the backrest **17** into an upright position. You will hear a click when the backrest has locked into its upright position. Fold out the side pads of the headrest **18** until you hear a click.

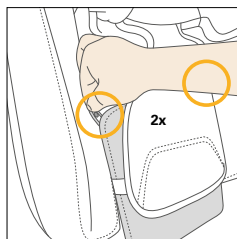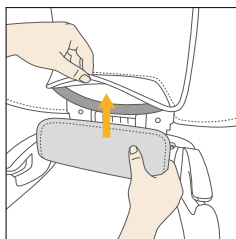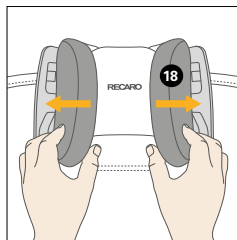
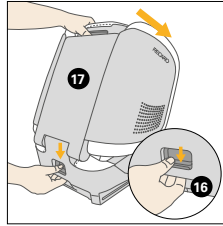
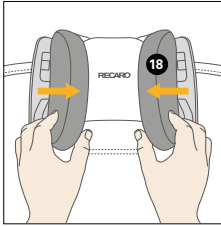
- Insert the lower backrest cover into the loop on the back of the upper backrest cover.
- Thread the elastic loops of the lower backrest cover onto the hooks on the left and right side of the backrest.

FOLDING THE CHILD SEAT

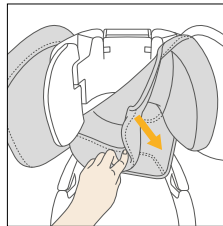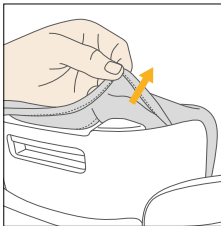
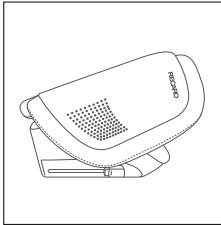
Fold in the side pads of the headrest **18** until you hear a click. Press down the button at the bottom of the backrest **16** and push the backrest **17** forwards towards the seating surface.

REMOVING THE SEAT COVER

The cover consists of 4 components fixed to the seat shell by elastic loops, snap buttons or brackets.

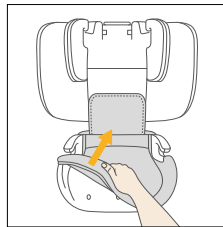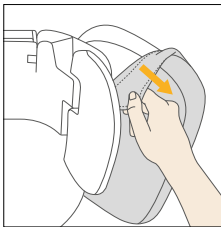- Remove the headrest cover by pulling the fabric out from the slots.
- Remove the side wings cover by pulling the fabric out of from the slots.
- Remove the seat cover by loosening the elastic loops from the back rest and then pull the fabric out of the slots.

WARNING! The child seat must under no circumstances be used without the cover.

CLEANING

It is important to use an original seat cover only since the cover is an essential part of the function. You may obtain spare covers at your retailer.

NOTE! Please wash the cover before you use it the first time. Seat covers are machine washable at max. 30°C on delicate cycle. If you wash it at higher temperature, the cover fabric may lose colour. Please wash the cover separately and never dry it mechanically! Do not dry the cover in direct sunlight! You can clean the plastic parts with a mild detergent and warm water.

WARNING! Please do not use chemical detergents or bleaching agents under any circumstances!

PRODUCT CARE

To guarantee that your child seat provides the maximum protection, it is essential to comply with the following points:

- All major components of the child seat must be regularly checked for damage. The mechanical parts must work without problems
- Make sure that the child seat is not jammed between hard parts such as the car door, seat rails etc., which can damage it
- If e.g. the child seat is dropped, it must be tested by the manufacturer before being used again

WARNING! The cover is an integral component of the child seat's safety design. The child seat must therefore never be used without the cover or with a cover of a different child seat.

WHAT TO DO AFTER AN ACCIDENT

WARNING! An accident may cause damage to the child seat that is not visible to the naked eye. Please exchange the seat without fail after an accident. In case of doubt contact the retailer or man

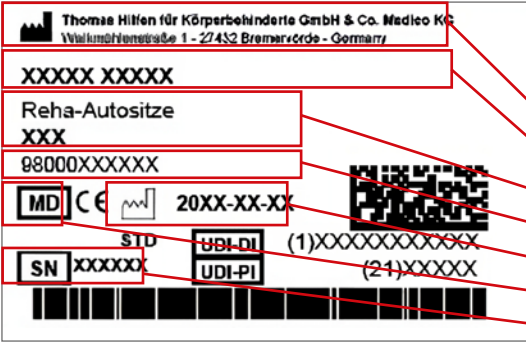
PRODUCT INFORMATION

If you have any questions contact your retailer first. You should have the following details ready:

- Serial number (see sticker)
- Make and model of the vehicle and the position of the vehicle seat on which the child seat is used
- Weight, age, and height of the child

Further information about our products can be found at www.recaro-kids.com.

4. Labels / Model Identification

Aufkleber	Bedeutung
 <p>Thomashilfen für Körperbehinderte GmbH & Co. Medico KG Walkmühlenstraße 1 - 27432 Bremervörde - Germany</p> <p>XXXXX XXXXX</p> <p>Reha-Autositze XXX</p> <p>98000XXXXXX</p> <p>MD CE 20XX-XX-XX</p> <p>STD UDI-DI (1)XXXXXXXXXXXX UDI-PI (21)XXXXXX</p> <p>SN XXXXXX</p>	<p>Barcode label (exchange image)</p> <ul style="list-style-type: none"> - Manufacturer - Product name - Product group - Item number short / long - MD = Medical Device - Date of manufacture - Serial number
 <p>Thomashilfen Walkmühlenstraße 1 27432 Bremervörde Germany</p> <p>Recaro Monza CFX Reha</p> <p>Nutzergewicht / user weight: 15 - 36 kg 33.1 - 79.4 lbs</p> <p>Nutzergröße / user size: 100 cm - 150 cm 39.3 inch - 59.1 inch</p> <p>Made in Germany</p> <p>CE i</p>	<p>Type plate Add picture</p> <ul style="list-style-type: none"> - Manufacturer - Product name - Max. user weight / max. load capacity - allowable user size - CE mark - Follow the instructions for use
 <p>Thomashilfen Walkmühlenstraße 1 27432 Bremervörde Germany</p> <p>Recaro Monza CFX Reha</p> <p>Festigkeitsnachweis durch Crashtest in Anlehnung an</p> <p>ECE R129/03 S Frontaufprall Test-Nr. S240321003 Seitenaufprall Test-Nr. S240321002</p> <p>Proof of strength by crash test in accordance to</p> <p>ECE R129/03 S Front impact Test-No. S240321003 Side impact Test-No. S240321002</p>	<p>Crashtest-Label</p> <p>Information: The product has via a strength verification in used on ECE R 129/03.</p>

5. Remarks

5.1 General Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- In some countries it is necessary to carry a medical certificate.
- The rehab car seat can be used on the back seats as well as on the front passenger seat. The chosen car seat **MUST** face forward.
- All labels have to remain on the product and should not be tampered with.

5.2 Safety Instructions

- Check at least every 3 months that the settings on the product still work for the user's current body measurements. If necessary, have a distributor / orthopaedic technician to adjust the settings.
- Do regular (weekly) checks to make sure that the setting options on the product work.
- Be careful of the moving parts when adjusting or folding the product.
- Only lift up the product by its firmly welded or screwed parts.
- Never use the seat's straps to lift or carry the seat.
- Before each use check that all the nuts, bolts, and screws are tight.
- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- Never leave the user unattended with the product.
- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- All users must be familiarised with the functions of the product. This means that the user can be quickly freed from the the product in an emergency
- The user should always be positioned with the five-point positioning harness (standard equipment). The seat and the user always have to be fastened with the vehicle's own three-point seat belt.

- Adjust the straps of the seat so that they fit closely to the user, but do not press. Make sure that the straps do not sag or twist.
- Before each journey, check that the product is securely attached.
- Even when the product is not in use, it must be properly secured with the vehicle's own 3-point safety belt. In the event of an accident, an unsecured seat can cause serious injury to the vehicle occupants.
- Intense sunlight and heat can lead to high surface temperatures on the product. As a result, there is a risk of injury when touching the product. Only use the product once it has cooled down enough. In general, try to protect the product from direct sunlight.
- Please note the safety standards which apply to your vehicle as outlined in the vehicle's Instructions for Use to ensure the safety of the user.
- The Instructions for Use have to remain with the product.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.
- The back stabiliser is mandatory when using the swivel function.
- If necessary, use a suitable aid to protect the vehicle seats from damage. Thomas-Hilfen accepts no liability for damage to your vehicle seats

6. Intended Purpose

6.1 Intended Use / Application Areas

The Recaro Monza CFX Reha car seat is a class 1 medical device.

It is designed for the transport of children with disabilities and for a safe seating position in motor vehicles. The seat is secured with the vehicle's own 3-point safety belt. The 5-point Harness of the rehab car seat is used to position and support the child in an upright sitting position.

The rehab car seat offers additional support for safe sitting, stabilisation of head control and a stable seat in the pelvic area. The product also offers a wide range of accessories for customisation.

Any other or additional use is considered improper.

6.2 Indication / Patient Target Group / Intended Users

The rehab car seat is used for children with structural and / or functional limitations or malpositions in the head and trunk area. It is also used for children who are mentally or neurologically impaired or have perceptual disorders. It provides a stable and physiological positioning when a standard child car seat is too small or not suitable or the vehicle's own safety belt is not sufficient for an upright sitting posture in the vehicle.

6.3 Risks Associated with Use and Contraindications

In order to find the right rehab car seat for the user, the attending physician should always be consulted. With proper use, risks can be excluded during application. Contraindications are not known.

7. Product Description

7.1 Scope of delivery

Your product is supplied pre-assembled in the basic configuration. Any accessories ordered are included with the product and must be installed subsequently. Instructions for use are enclosed with the product.

7.2 Basic equipment

Seat and back with integrated headrest in 11 different positions; Isofix connection; attachable side impact protection with energy-absorbing elements; removable head, thorax and seat side pads; 5-point Belt; shoulder belt pads; Crotch pad; removable abduction guide; removable covers.

7.3 Commissioning

- Carefully remove the transport locks and packaging.
- Make sure that packaging material and small parts cannot get into the hands of children small parts from getting into the hands of children to avoid choking or the risk of suffocation.

7.4 User training

The user is instructed in the use of this aid by authorised specialist personnel with reference to these instructions for use.

Car seats only (installation with specialist adviser) Remains in place

The technical advisor should install the seat in the vehicle together with the user and adjust the seat to the respective dimensions of the current user.

It must be ensured that the user has been instructed in all functions of the product as well as the installation and removal from the vehicle.

8. Adjustment options / performance features

8.1 5-point harness

Please note:

The 5-point harness is used to secure users <1m tall and must be used. For users taller than 1 metre, it is used to position the child in the car seat and does not protect them in the event of an accident. In this case, the 3-point safety belt in the car must always be used to secure the child in the seat.

8.2 Changing the belt length:

There are two different attachment eyelets in the upper area of the shoulder straps to attach the upper shoulder straps to the headrest at a suitable length.

Pull on the loose harness straps to shorten and tighten the harnesses. To lengthen the belts, press the belt tensioner release button and then pull the harness to the desired length. By changing the adjustment range on the belt tensioner, you can adapt the Harness even better to the user. The crotch strap can be adjusted in length. This allows the position of the belt buckle to be customised to the user.

8.3 Strap fastener:

Press the red indentation on the buckle to open it.

To close the Harness, insert the two upper belt holders into the buckle and press it together until it engages with an audible 'click'.

Fig. 1: Opening the belt buckle ('click')

8.4 Head cover

The head cover is fitted with a soft Velcro velour fabric on the inside. This allows additional padding to be attached if necessary. In addition, the cover has 2 sewn-in pockets which can be filled with wadding or other filling material if required. DO NOT use any hard, sharp and/or pointed objects for this purpose.

8.5 Lateral trunk support (Fig. 2)

The thorax pad angles are attached to the back. To change the position, loosen the Backrest upholstery. The Velcro connection can then be loosened and the Lateral trunk supports can be reattached in the desired position. Then close the Backrest upholstery again and attach it to the side hooks using the elastic bands.

8.6 Seat side pads (Fig. 3)

The seat side supports are attached to the side guides and fixed at the back of the seat with the fastening strips. behind the seat with the fastening strips

8.7 Pelvic guide (Fig. 3)

The pelvic pads provide the greatest possible support and improve the sitting posture for very narrow users.

8.7 Abduction block (Fig. 4)

The abduction guide is guided forwards with the belt strap through the seat cushion and fastened with the press stud.

9. Accessories

Note: Information on fitting can be found in the instructions for use supplied with the accessories. If the instructions for use are missing, you can download them from www.thomashilfen.de. The accessories are fitted / removed using standard tools.

9.1 Adapter plate (Fig. 5)

The adapter plate is a prerequisite for using other accessories. This is screwed under the rehab car seat using the 4 screws.

9.2 Swivel function (Fig. 6)

With the swivel function, the rehab car seat can be swivelled towards the vehicle door. To do this, press the red release lever and turn the seat towards the door. This makes it easier for carers to transfer the user and access the seat's 5-point Harness. It is easier for the user to get in and out of the seat independently.

Attach the swivel function under the adapter plate of the rehab seat.

9.3 Tilt function (Fig. 7)

The tilt function makes it easier to optimise the positioning of the user if control of the upper body or head is restricted. The tilt function allows the 'Nextkid Rehab' to be tilted backwards in 2.5° increments.

To set the desired tilt, loosen the two knurled screws on the side. Change the tilt and secure the tilt plate again with the knurled screws. The tilt function can be adjusted from 0° to 10°.

9.4 Rotation and tilt function (Fig. 8)

The turn and tilt function combines the two functions in one accessory. (swivel function and 10° tilt function)

9.5 Footrest adapter (Fig. 9)

The footrest adapter is used to attach a footrest or a support foot if no swivel, tilt or swivel and tilt function is used.

9.6 Stabilisation bar (Fig. 10)

If your vehicle does not offer an Isofix connection, the stabilisation bar can be used as an alternative. The stabilisation bar is pushed into the holder under the seat plate from behind. Loosely screw the enclosed screws into the holder so that the stabilisation bar can still be moved.

Determine the required position of the stabilisation bar: To do this, place the seat in the vehicle and guide the stabilisation bar between the seat surface and the backrest of the vehicle seat. The backrest of the rehab seat must have as much contact as possible with the backrest of the vehicle. Mark the desired position at the transition between the stabilisation bar and the mount under the seat plate. Remove the rehab seat from the vehicle and secure the stabilisation bar at the marked position.

Depending on the vehicle seat, the stabilisation bar can also be used upside down. To do this, fit the bracket with the bend facing either upwards or downwards.

9.7 Footrest (Fig. 11)

The footrest provides support for the user's feet. The footrest is folded up for getting in and out of the wheelchair. A magnet secures this position of the footrest. The footrest is pushed into the holder from the front. Adjust the depth of the footrest to the seat in the vehicle. Screw the footrest to the sides of the carrier using the screws provided.

The desired angle can be set using the red eccentric levers on both sides. Ensure that the eccentric levers are firmly tightened after adjustment. Select the height of the footrest legs so that they support the rehab seat securely in the footwell of the vehicle. The height can be adjusted using the side screws. Ensure that the Allen screws are present on the footrest. The final adjustment is made by adjusting the footrest feet. It is essential to lock the threaded rod with the plastic lock nuts!

If the footrest is not at the correct height for the user, loosen the screws on both sides of the footrest. Adjust to the correct height for the user and retighten the screws.

9.8 Support leg (Fig. 12)

The support leg provides support for the rehab seat in the footwell of the vehicle. The support leg is pushed into the holder from the front. Adjust the depth of the support leg to the seat in the vehicle. Tighten the support leg with the enclosed screws.

The desired angle can be set using the red eccentric levers on both sides. Ensure that the eccentric levers are firmly tightened after adjustment.

Select the height of the support legs so that the rehab seat is securely supported in the footwell of the vehicle. The height can be adjusted using the threaded rods. It is essential to lock the threaded rod with the plastic adjuster after adjusting the height.

9.9 Underarm support (Fig. 13)

The lower arm support is placed on the side elevations of the rehab seat. The short harnesses on the sides of the table are routed to the rear and attached to the D-rings of the 5-point harness. The downward-facing straps are attached to the fleece strips on the seat shell. Then adjust the belt length so that your child can make good use of the underarm support and it does not pinch your child.

9.10 Back stabilizer (Fig. 14)

The back stabilizer is mandatory when using the swivel function.

Use without it can lead to damage to the rehab seat or, in the worst case or, in the worst case, injury to the child.

To install the back stabilizer, attach it to the back of the rehab car seat using the enclosed screws in the hollow anchors located there. This Accessory overrides the folding mechanism.

9.11 Belt connector (Fig. 15)

The Harness connector is designed to prevent the user from wriggling out of the 5-point harness during the journey.

To fit the belt connector, loosen the belt straps from the plastic buckles. (Do not loosen the screw points!) The belt connector can then be threaded onto the two exposed belt ends. Then reattach the free strap ends to the plastic buckles.

The belt connector must always be fastened below the plastic buckles!

9.12 Adjustable abduction guide (Fig. 16)

The abduction guide is attached to the adapter plate and can be adjusted in height and depth. It can only be used in combination with the adapter plate.

9.13 Seat depth extension (Fig. 17)

The seat depth extension is used to extend the seat surface for taller users and thus maximise seating comfort. The seat depth extension is attached to the adapter plate and can be mounted at any depth. It can only be used in combination with the adapter plate.

9.14 Seat wedge (Fig. 18)

The Seat wedge makes it easy to tilt the seat by 10°. The angle is not adjustable.

9.15 Anti-escape waistcoat (Fig. 19)

The anti-escape waistcoat prevents the child from wriggling out of the car seat belt. Dangerous gaps between the belt and the child are covered, which the child would otherwise use to pull the belt off the shoulders.

9.16 Car seat protector with back

The Seat wedge makes it easy to tilt the seat by 10°. The angle is not adjustable.

Anti-escape waistcoat (Fig. 20)

The anti-escape waistcoat prevents the child from wriggling out of the car seat Harness. Dangerous gaps between the harness and the child are covered, which the child would otherwise use to pull the harness off the shoulders.

9.17 NapUp

If the user requires a head fixation system, you can attach the NapUp inside the headrest with the aid of the NapUp installation instructions.

10. Technical Data

Recaro Monza CFX Reha

Seat depth (to crotch strap)	22 cm / 8.7"
Seat depth (front edge)	33 cm / 13"
Seat width	27 cm / 10.6"
Back height (seat inside)	55 - 72 cm / 21.7 - 28.3"
Shoulder height (belt guide)	31 - 51 cm / 12.2 - 20.1"
Shoulder width (basic model)	35 - 45 cm / 13.8 - 17.7"
Seat tilt (with tilt plate)	2,5° / 5° / 7,5° / 10°
Overall dimensions (w x h x d)	45 x 65 x 40 cm / 17.7 x 25.6 x 15.7"
Weight of the seat (basic model)	8,5 kg / 18.7 lbs
User height	100 - 150 cm / 39.4 - 59.1"
Max. user weight	36 kg / 79.4"

11. Material Specifications

The upholstery material used is "flame-retardant", tested in accordance with DIN EN 1021-1 and DIN EN 1021-2. The degree of resistance to ignition of materials and assemblies was tested in accordance with the standard and assessed as harmless. The 100% polyester fabric is a high-quality, breathable fabric and has been positively tested for biocompatibility. (DIN EN ISO 10993-5 and 10993-10)

12. Cleaning, Maintenance, and Repair

12.1 Cleaning, Care Instructions, and Disinfection on a Daily Basis

Padding

The padding can be easily removed in a few simple steps and washed separately. The padding consists of a fabric cover

and sometimes fillers made of foam. Check if the padding has a zipper or a hook and latch fastener. If it does, open the

padding and remove the foam filler before washing the cover. Please also note the following care instructions.

We would like to point out that even high-quality materials can fade as a result of long and intense exposure to sunlight or frequent washing.

When changing users, a “padding set for reuse” is offered by the manufacturer.

12.1.2 Hook and Loop Fasteners

To keep the hook and loop fasteners working well, clean them now and then with a brush. Always close or cover the hook and loop fasteners before washing.

12.1.3 Plastic Parts

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

12.1.4 Metal Parts

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

12.1.5 Disinfection

Clean the padding, plastic parts, and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

12.2 Maintenance and inspection by the distributor

12.2.1 Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test is. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user’s responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

12.3 Maintenance Schedule – Annual Inspection

	Area	Function		Damage		Replace	
1	Kunststoffkorpus	👁		👁			
2	Adjustment options	👁		👁			
3	Padding			👁			
4	Belts	👁		👁			
5	Plastic parts	👁		👁			
6	Screws and nuts	👁		👁			
7	Labels			👁			
Accessory							
8	Padding			👁			
9	Adjustment options	👁		👁			
10	Plastic parts	👁		👁			
11	Metal parts	👁		👁			
12	Screws and nuts	👁		👁			
done by:				on:			
👁 = Inspection ↻ = Reuse ✓ = Test completed							

12.4 Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at www.thomashilfen.de. To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

12.5 Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

12.6 Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

13. EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

14. Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

15. Service life / re-use

The product has a theoretical service life of up to 6 years with proper and regular maintenance regular maintenance based on the manufacturer's instructions. The product can be used for up to 2 years beyond beyond this period if it is in a safe and technically perfect condition technically perfect condition. Periods of storage at a specialist dealer are excluded from the useful life.

The theoretical service life is not a guaranteed service life. It depends on the intensity of use and is always subject to a case-by-case assessment by the specialist dealer.

The service life does not include wearing parts such as upholstery, covers, etc,. The product is not suitable for re-use.

16. Distributor

Astris PME Pty Ltd

Young & Canberra Office:
Astris PME Pty Ltd
5 McVeigh St
Young NSW 2594

Sydney Office:
Astris PME Pty Ltd
30 Prime Dr
Seven Hills NSW 2147

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Tendercare Ltd.
Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG